

# EACH EAR CANAL IDENTIFICATION GUIDE™

## About Ear Canals

Ear canals are usually angled (sometimes sharply) in one direction or another, include various bends, and can be obstructed by cartilage near the canal opening.

## About Your *Each Ear Canal Identification Guide*™

Your *Caring Industry Leader* has selected the illustrations that most closely identify the angles and bends of each of your ear canals and has provided you with this *Each Ear Canal Identification Guide*™ to help you effectively aim and insert your hearing aid speakers using *Gruv Buttons*™.

Patient Name

Practitioner Name

Business Name

Address

Address

Phone

## FIRST BEND (1 - 6)



INITIAL ANGLE (A - I)



LEFT EAR	1 Up	2 Level	3 Down	4 Front	5 Back	6 No Bend	Or
A Level							J. Straight Up and Front
B Up							K. Straight Sharply Up and Front
C Sharply Up							L. Straight Down and Front
D Down							M. Straight Sharply Down and Front
E Sharply Down							N. Straight Up and Back
F Front							O. Straight Sharply Up and Back
G Sharply Front							P. Straight Down and Back
H Back							Q. Straight Sharply Down and Back
I Sharply Back							

RIGHT EAR	1 Up	2 Level	3 Down	4 Front	5 Back	6 No Bend	Or
A Level							J. Straight Up and Front
B Up							K. Straight Sharply Up and Front
C Sharply Up							L. Straight Down and Front
D Down							M. Straight Sharply Down and Front
E Sharply Down							N. Straight Up and Back
F Front							O. Straight Sharply Up and Back
G Sharply Front							P. Straight Down and Back
H Back							Q. Straight Sharply Down and Back
I Sharply Back							